



Installing ABPM Software

Step 1. Visit www.cardiostudy.ca


Step 2. In the top right corner of the website, click on Downloads

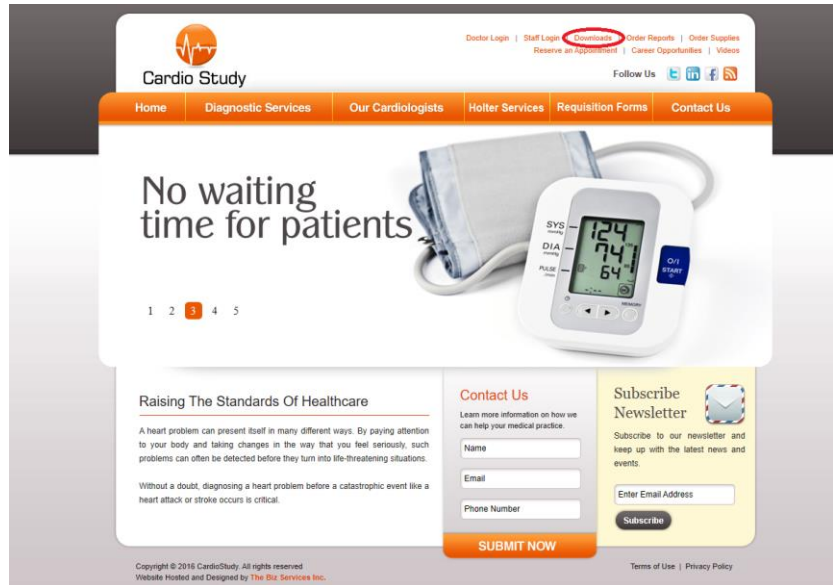



Step 3. On the next page, click to download "ABPM Software"

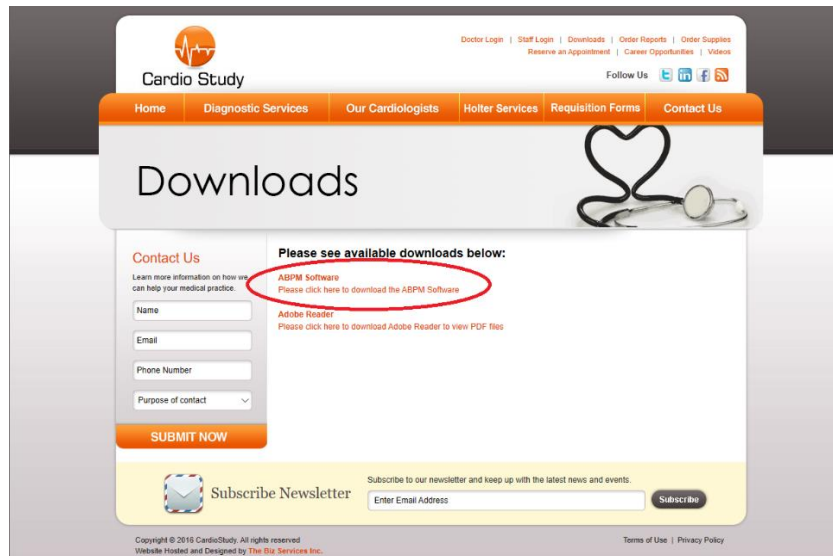


Step 4. If prompted to "Run or Save" the file, click on Run. **Otherwise** click "Save" and open the file once it is downloaded.

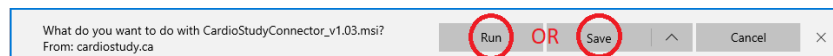


Step 5. During installation, you may be prompted by your computer to allow the program to install. In this event, click on "More Info", followed by "Run Anyway"

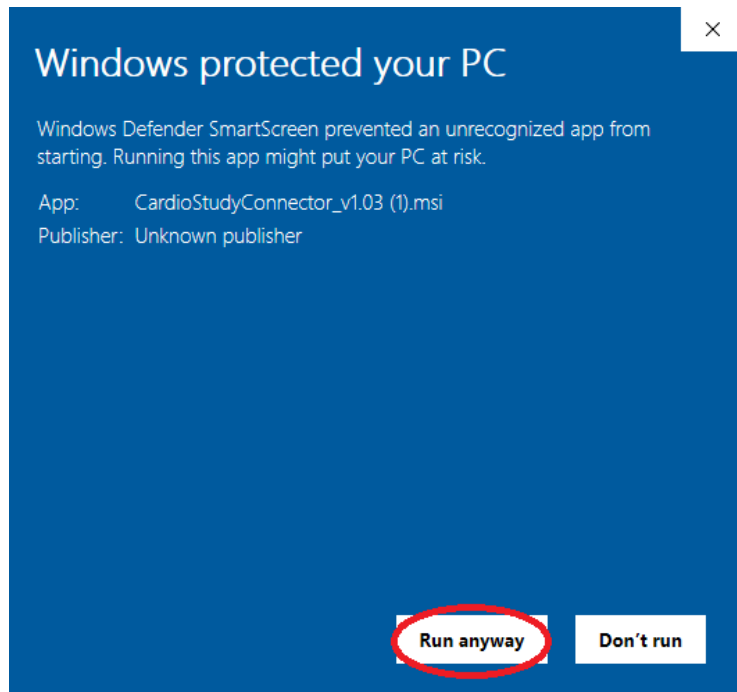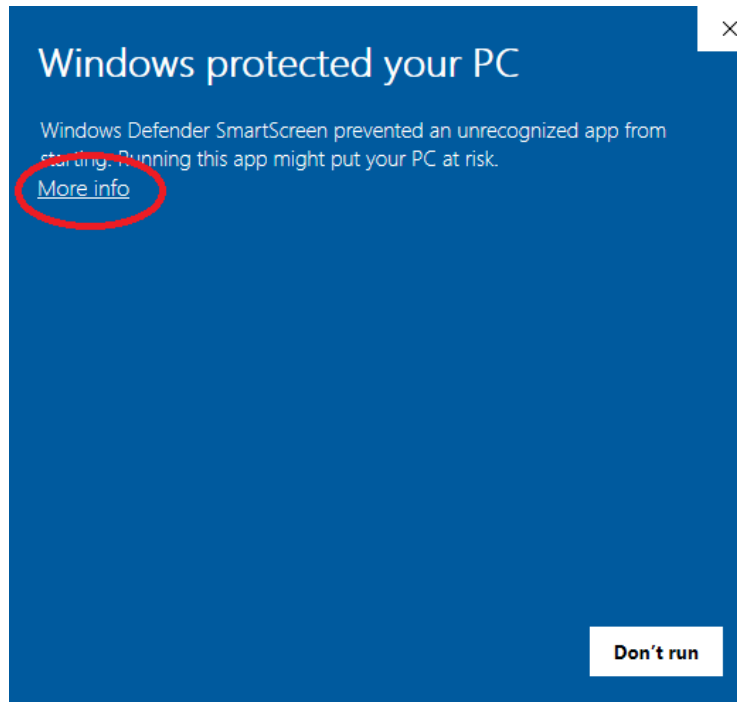


Step 6. Next you will see the installation wizard pop up. Click on "Next"

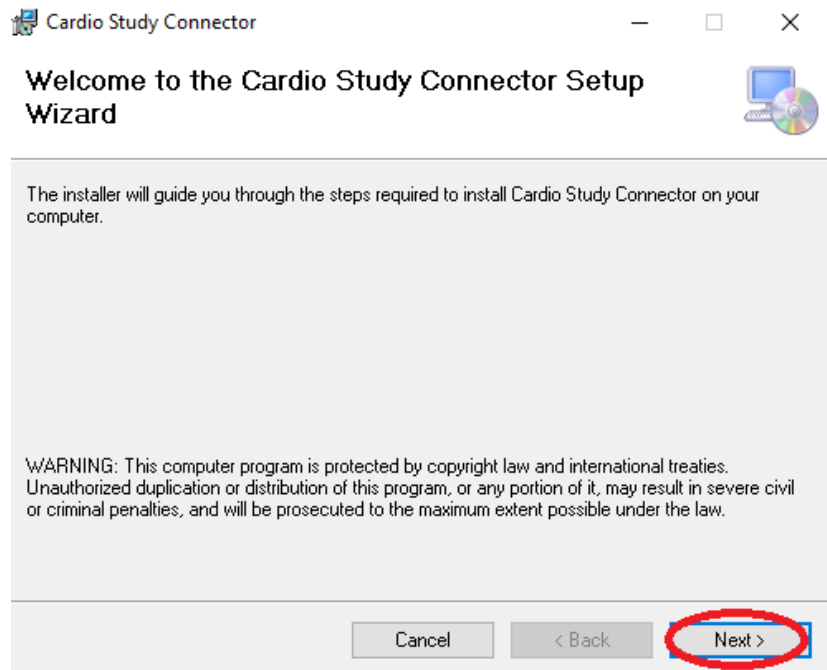


Step 7. On the next screen, select "Everyone", then click "Next".

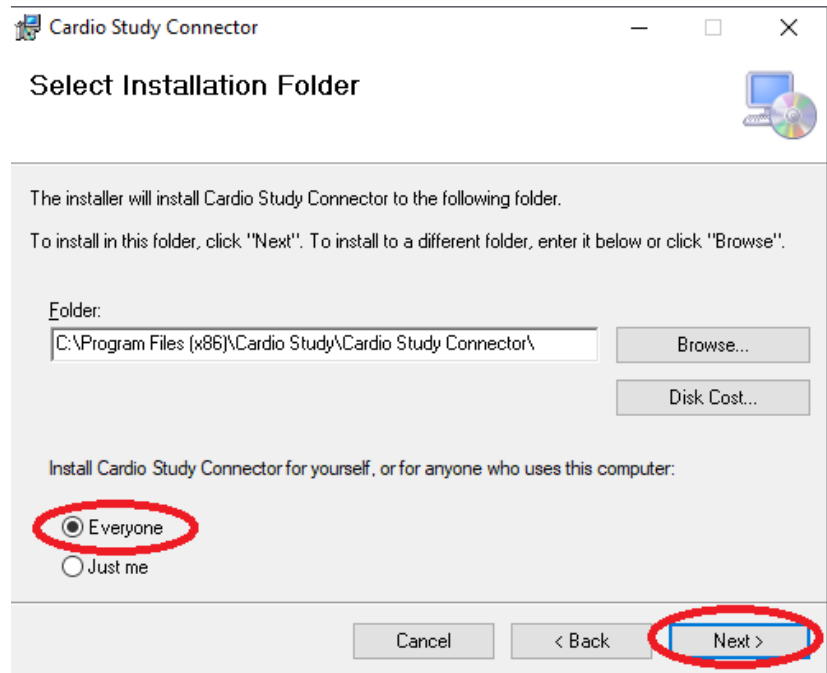

Step 8. On the next screen, click “Next”.

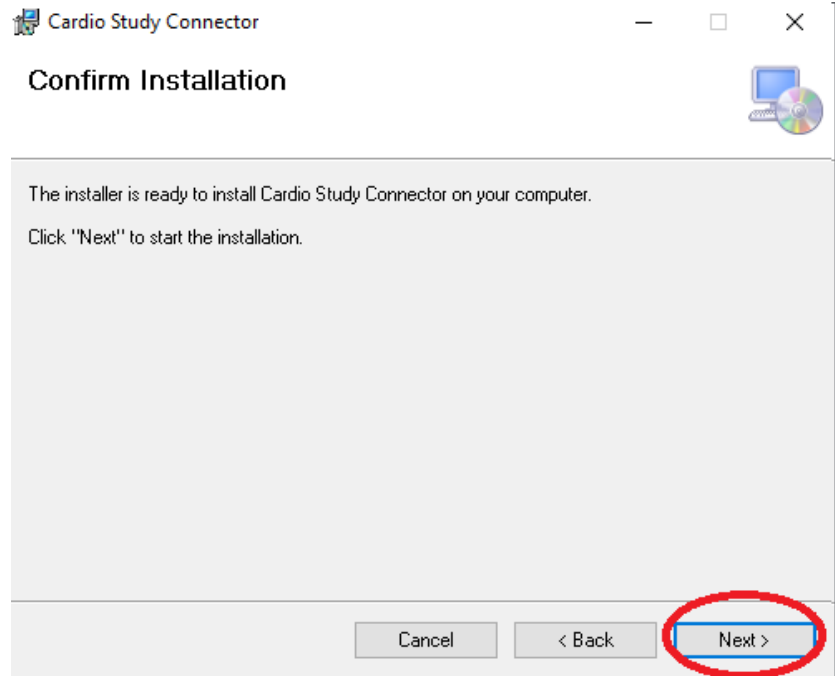

Step 9. You may be prompted by your PC to grant permission to install the file. Click “Yes”.

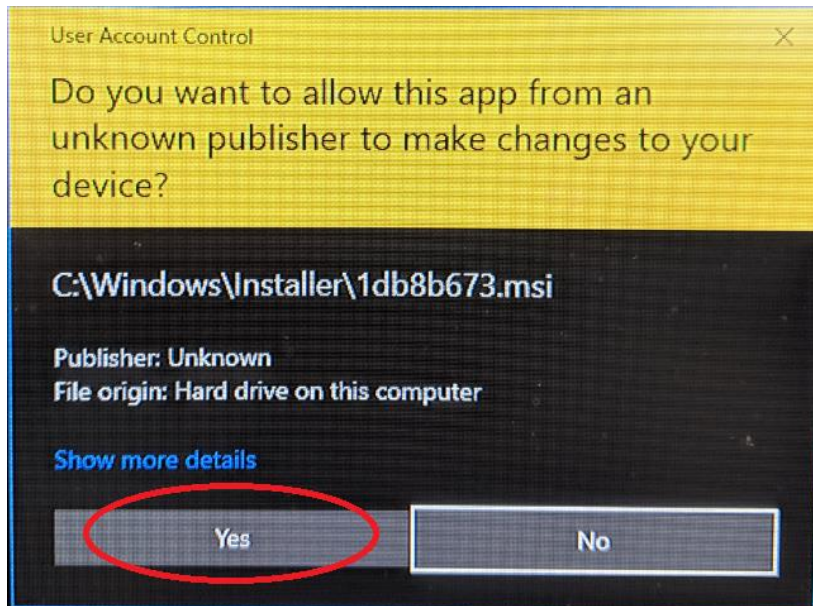

Step 10. Once the installation is complete, you may click "Close".

